

Appendix for Hybrid Closed Loop Insulin Pump - Medtronic 670G/770G

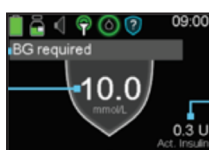
Use this appendix in conjunction with the student's Diabetes Action and Management Plan.

STUDENT'S NAME

GRADE / YEAR

INSULIN DELIVERY**AUTO MODE**

Blue shield on screen indicates the pump is in **AUTO** mode. The pump will alert the student that action is required when in AUTO mode.



Grey Shield - Safe Basal becomes active when the pump requires more information e.g. a Blood Glucose Level (BGL) check.

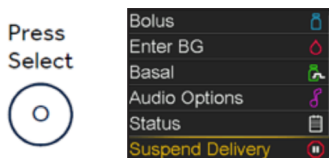
Auto mode / Safe basal means basal insulin delivery is informed by a continuous glucose monitor (CGM). The student **MUST** respond to all pump commands. If the pump request has not been attended to, it will switch to **MANUAL** mode.

**MANUAL MODE**

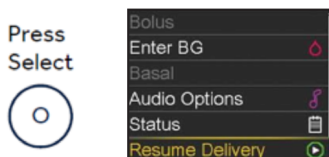
No Blue or Grey shield seen on screen means the pump has switched to **MANUAL** mode. Insulin is still being safely delivered.

EATING AND DRINKING

The student **MUST** bolus for all carbohydrate food and enter a **BLOOD** glucose reading in both auto and manual mode.

PHYSICAL ACTIVITY**DISCONNECTING PUMP BEFORE SPORT**

Select '**Suspend Delivery**' from the main menu by pressing the 'select' button and scrolling down to the **SUSPEND DELIVERY** option

**RECONNECTING PUMP AFTER SPORT**

Reconnecting pump select '**Resume delivery**' from the main menu by pressing the 'select' button and scrolling down to the **RESUME DELIVERY** option.